




NAME	ROUND BUR	TAPERED BUR	FOOTBALL BUR
ITEM NO.	70-1014-BRX0018	70-1014-BRX0019	70-1014-BRX0016
SHANK	FG	FG	FG
SIZE	018	021	023

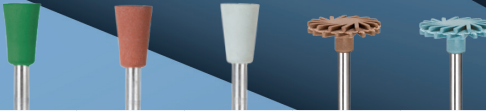

NAME	GREEN CUP	BROWN CUP	WHITE CUP	BROWN DISC	BLUE DISC
ITEM NO.	70-1014-BRX0017	70-1014-BRX0011	70-1014-BRX0012	70-1014-BRX0022	70-1014-BRX0021
SHANK	RA	RA	RA	RA	RA

The BruxZir™ Adjustment & Polishing Kit equips you with the tools to modify BruxZir® restorations and achieve the ultimate high shine and lifelike vitality.

CLEANING AND STERILIZATION INSTRUCTIONS

Refer to IFU-12331 for detailed cleaning and sterilization instructions.



Prismatik Dentalcraft, Inc.
2144 Michelson Drive
Irvine, CA 92612, USA

888-303-3975 • glidewelldirect.com

UM-6415_6 GD-458-051426

BruxZir™ Adjustment & Polishing Kit



By Prescription Only



Consult Instructions for Use

BRUXZIR ADJUSTING BURS

Always use light pressure with water and air spray when adjusting BruxZir® restorations to avoid microfractures.



A **round bur** is used for adjusting a fossa.



A **tapered bur** is most effective for adjusting cusps or proximal contacts.



A **football-shaped bur** is most effective for adjusting the occlusal surfaces of posterior teeth and the lingual surfaces of anterior teeth.

BRUXZIR POLISHING CUPS



Using light pressure and no water, begin pre-polishing with the **green cup** to remove abrasions left by the diamonds during adjustments. The green cup can also be used alone for minor adjustments if you don't want to use a diamond.



Continue pre-polishing with the **brown cup** until a more glossy look appears on the adjustment areas.



Use the **white cup** with light to medium pressure to achieve a “wet” high shine.

BRUXZIR POLISHING DISCS



The **brown disc** gently removes surface defects. Use in circular motions at 8,000–15,000 RPM to make minor adjustments to occlusal surfaces without affecting the anatomy.



The **blue disc** is ideal for difficult-to-reach areas. Use light to medium pressure at 8,000–15,000 RPM to achieve a natural, high-shine finish in as little as 20 seconds.

Device Type	Recommended Speed (RPM)	Max Speed (RPM)
Round Bur	45,000 to 85,000	300,000
Tapered and Football Shaped Bur	40,000 to 75,000	300,000
Polishing Cups (White, Green, Brown)	10,000 to 15,000	15,000
Polishing Discs (Brown, Blue)	8,000 to 15,000	15,000

Note: Excessive pressure will lead to overheating and premature degradation of the instruments, as well as possible damage to the pulp tissue. Expected life of adjusters and polishers are 10–15 uses depending on the substrate and applied pressure.